**Step 1**      **Inserting the Feeding Set**

A. For a 500-mL feeding bag:  
Place the bag in the first large section of the backpack. (Fig. 1)  
Thread the end of the tubing through the middle set of Velcro straps, and through the hole on the bottom, toward the back of the backpack. The end of the tubing should exit the back of the backpack. (Fig. 1)

B. For a 1,200-mL bag:  
Place the bag in the largest section, to the rear of the backpack. (Fig. 3) Thread the end of the tubing through the Velcro straps on the bottom, and out the back of the backpack. (Fig. 2) Thread the cassette through the hole toward you until it sits in the pump section.

**Step 2**      **Securing the Feeding Bag**

Squeeze any air out of the feeding bag and pour desired amount of formula into the bag. Close it tightly.  
Secure the neck of the feeding bag with the Velcro strap inside the backpack. Secure the handle of the bag with the inner metal clip. (Fig. 3)

**Step 3**      **Inserting the Pump**

Load the cassette into the pump and close the door. Prime the tubing per pump instructions and set the pump to the desired setting. Secure the pump in the front section of the backpack with the Velcro straps. Be sure that the pump fits snugly. Close all sections of the backpack, using the zippers.

**Step 4**      **Making the Backpack Comfortable and Secure**

Adjust the shoulder straps according to comfort. Use the abdomen strap to secure the backpack around the waist.

Fig. 1

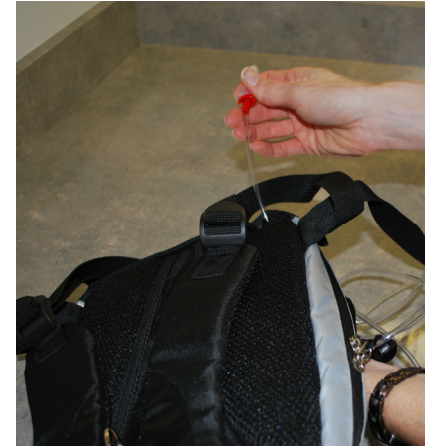
Fig. 2

Fig. 3

Fig. 4

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